

POINT MATE (Acu-Point Finder) INSTRUCTIONS:

The Point Mate does not come with any instruction booklet, however it works as follows:

There is a grounding pole (basically a metal rod) that is attached to a wire that's plugged into the small base unit. There is also a point locator probe attached to a wire, which is plugged into the small base unit.

Make sure you have good batteries in your Point Mate, turn it on and then hold the grounding pole (make sure it is in contact with the skin somewhere on the body of the person being treated) or have the patient hold it and then search for the point with the pointed locator probe attachment. The louder the sound and the brighter the light, the closer you are to an acupuncture point, which is picked up due to skin heat resistance. An acupuncture point is like a small explosion of heat beneath the surface of the skin.

The temperature of the skin and the amount of moisture on the skin affects the sensitivity of the device. Colder dryer skin requires higher sensitivity (adjust dial from 1-10), warmer moist skin requires lower sensitivity.

I trust this will help you.

Best Regards,

Kalon Prensky

www.healthiswealthmaui.com

and

www.quantum-healing-lasers.com